Heizkissen HKS / HKF / HKC / HKM / HKN

Heat Pad HKS / HKF / HKC / HKM / HKN

Coussin chauffant HKS / HKF / HKC / HKM / HKN

Termoforo HKS / HKF / HKC / HKM / HKN

Cojín Eléctrico HKS / HKF / HKC / HKM / HKN

Almofada térmica HKS / HKF / HKC / HKM / HKN

Verwarmingkussen HKS / HKF / HKC / HKM / HKN

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Read the following safety instructions carefully before using the heated cushion.

Keep the instruction manual in a safe place for later use.

Do not puncture the heated cushion.

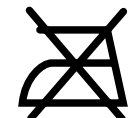
Do not use the heated cushion when it is folded.

Only use the heated cushion indoors.

Do not chlorinate!

Do not iron the heated cushion.
You may iron the cushion cover at a low temperature.

Do not dry clean!

Do not dry the heated cushion in a tumble drier.
You may dry the cushion cover in a tumble drier.

Do not wash the heated cushion.
You may wash the cushion cover in a normal wash at max. 40 °C.

1.1 Thank you very much

Thank you for your confidence in us and congratulations!

You have purchased the Heat Pad **HKS / HKF / HKC / HKM / HKN**, a quality product by **MEDISANA**.

To ensure the best results and long-term satisfaction with your **MEDISANA** Heat Pad, we recommend that you read the following operating and maintenance instructions carefully.

Please read the following safety information carefully before using the device and keep these instructions in a safe place for later use.

1.2 Important

- Carefully check the heat pad before each use.
- Do not use the pad if there is visible damage to the pad, controls or leads.
- Before connecting the device to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- Do not fold or crease the heat pad during use.
- Neither pierce the pad with, nor attach safety pins or other spiky or sharp objects to the pad.
- Do not use the heat pad on children or persons who are disabled, asleep or insensitive to heat.
- Never use the pad unsupervised.
- The unit is designed for home use only.
- Do not fall asleep while the heat pad is switched on.
- Prolonged use of the heated cushion at a high setting could lead to skin burns.
- Do not cover the pad with any other cushions. The control switch should not be covered or placed on or below the pad while the unit is in operation.
- Do not sit on the heat pad. Instead, rest the pad on or against the part of the body to be treated.
- Never touch a heat pad that has fallen into water. Unplug the unit from the mains outlet immediately.
- Keep the mains lead away from hot surfaces.
- Never carry, pull or turn the heat pad by the mains lead and do not let the lead get tangled.
- Do not use the heat pad when wet and only use it in dry environments (not in the bathroom or similar environments).
- The controls and leads must not be exposed to any kind of moisture.
- If a fault occurs, do not attempt to repair the unit yourself. Repairs must only be carried out by an authorized specialist dealer or other suitably qualified personnel.

1.3 Health tips and recommendations

- Should you have any concerns about health issues, consult your doctor before using the heat pad.
- Do not use the heat pad on parts of the body that are swollen, injured or inflamed.
- Should you experience persistent pains in the muscles and joints, please inform your doctor. Persistent pains can be a symptom of serious illness.
- If you experience any pain or discomfort while using the unit, stop using it immediately.

2.1 Items supplied and packaging

Please check first of all that the unit is complete.

The following parts are included as standard:

- 1 **MEDISANA HKS / HKF / HKC / HKM / HKN** Heat Pad with cover
- 1 Fastening strap (included with the **HKN**)
- 1 Instructions for use

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING

To avoid risk of suffocation, keep packaging film away from children!

2.2 Operation

You can use the **MEDISANA HKS / HKF / HKC / HKM / HKN** heat pads to gently warm and relax various parts of the body. Using the heat pad stimulates blood circulation in hardened muscles and helps you to relax after a strenuous day.

The heat pad features an electric temperature control that regulates the temperature according to the chosen setting. With the hook and loop strap supplied with the pad you can position and fit the pad to the desired part of the body.

Only use the heat pad in the cover supplied with it.

2.3 HKS Heat Pad

Plug the unit into the mains outlet and move the slider control from position **0** to position **1**. The power LED will light up in red. After a few minutes you will be able to feel the pad getting noticeably warmer. If you require a higher temperature setting, move the switch to the next positions, **2** or **3**, or to the maximum heat setting, position **4**. If the pad gets too hot for you, move the slider back to position **3**, **2** or **1**.

To switch the unit off, move the switch back to position **0**. The red power LED will go out, indicating that the unit is switched off. Unplug the unit from the mains outlet.

The heat pad will switch off automatically after approx. 90 minutes of continuous use. To switch it back on again, move the slider control to position **0** and then back to the desired heat setting.

Unplug the unit from the mains outlet once you have finished using it.

2.4 HKF, HKC, HKN Heat Pad

Plug the unit into the mains outlet. The red LED indicates that the heat pad is connected to the mains supply. Now press the ●-button and switch the unit on. The first segment of the LED display for the lowest temperature setting will light up in red. By pressing the ▲-button you can select the higher temperature settings, **2** to **6**. The selected temperature setting is indicated by the number of lit segments. When setting **6** is selected, press the ▲-button once more and the heat pad will return to temperature setting **1**.

You can switch the heat pad off using the ●-button. If you switch it on again, the previously selected temperature setting will still be stored, so long as the unit remains plugged into the mains outlet.

The heat pad will switch off automatically after approx. 90 minutes of continuous use. To switch it back on again, press the ●-button.

Unplug the unit from the mains outlet once you have finished using it.

Additional information for the HKF model

The **HKF** heat pad features a cover with 2 compartments, one for heat pads and the other for a foam inlay. You can remove this inlay from the cover and then moisten it (e.g. using a spray bottle) before putting it back in the cover. You can then use the heat pad with the moistened side resting on the desired part of the body, providing a more intensive heat treatment.

Please remember that the inlay must only be moistened, not wet or soaked. Under no circumstances should you immerse or soak the pad in water.

Additional information for the HKN model

This specially-shaped back and neck pad is especially designed for use on these areas of the body. It is individually adjustable to your body thanks to hook and loop fasteners in the neck and hip areas.

3.1 Cleaning and maintenance

Heat pad

- Before cleaning the pad, unplug it from the mains outlet and let it cool down for at least ten minutes.
- Clean the **HKS** and **HKF** (PVC-surface) heat pads using a soft, damp cloth. Do not let water enter the heat pad or the control unit. After cleaning, dry off the unit with a dry cloth or towel. Only use the unit again when it is completely dry.
- The **HKC**, **HKM** and **HKN** (cotton surface) heat pads can be cleaned with a brush when dry.
- Never use strong detergents or hard brushes.
- You should untangle the cable from time to time.
- Store the unit in a clean and dry place, laid out flat with nothing covering or resting on top of it.

Heat pad cover

- Remove the heat pad from its cover.
- Dry clean the removable cover or as detailed in the instructions on the label.

3.2 Disposal

This product must not be disposed together with the domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority or your dealer for information about disposal.

3.3 Specifications

Name and model

MEDISANA Heizkissen **HKS / HKF / HKC / HKM / HKN**

Model	HKS	HKF	HKC / HKM	HKN
Voltage, frequency	: 230-240 V, 50 Hz	230-240 V, 50 Hz	230-240 V, 50 Hz	230-240 V, 50 Hz
Heat output	: 60 Watts	60 Watts	60 Watts	60 Watts
Dimensions	: 330 x 400 mm	360 x 440 mm	360 x 440 mm	400 x 650 mm
Weight :	480 g	480 g	480 g	560 g
Mains lead length	: 2,80 m	2,80 m	2,80 m	2,80 m
Operating conditions	Only use in dry rooms as described in the operating instructions			
Storage conditions	Laid out flat, clean and dry			
Art.-No.	: 60102	60132	60112 / 60142	60122
EAN	: 4015588601026	4015588601323	4015588601125 / 4015588601422	4015588601224

In accordance with our policy of continual product improvement, we reserve the right to technical and design changes without notice.

4.1 Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to send in the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

MEDISANA AG

Itterpark 7-9
D40724 Hilden

Tel.: +49 (0) 2103 / 2007-60
Fax: +49 (0) 2103 / 2007-626
eMail: info@medisana.de
Internet: www.medisana.de

The service centre address is shown on the last page.

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D&L Marketing Oy
ALERE Electronics AB